



# IV GLOVE 2.0



## FEATURES, SPECS AND INSTRUCTIONS

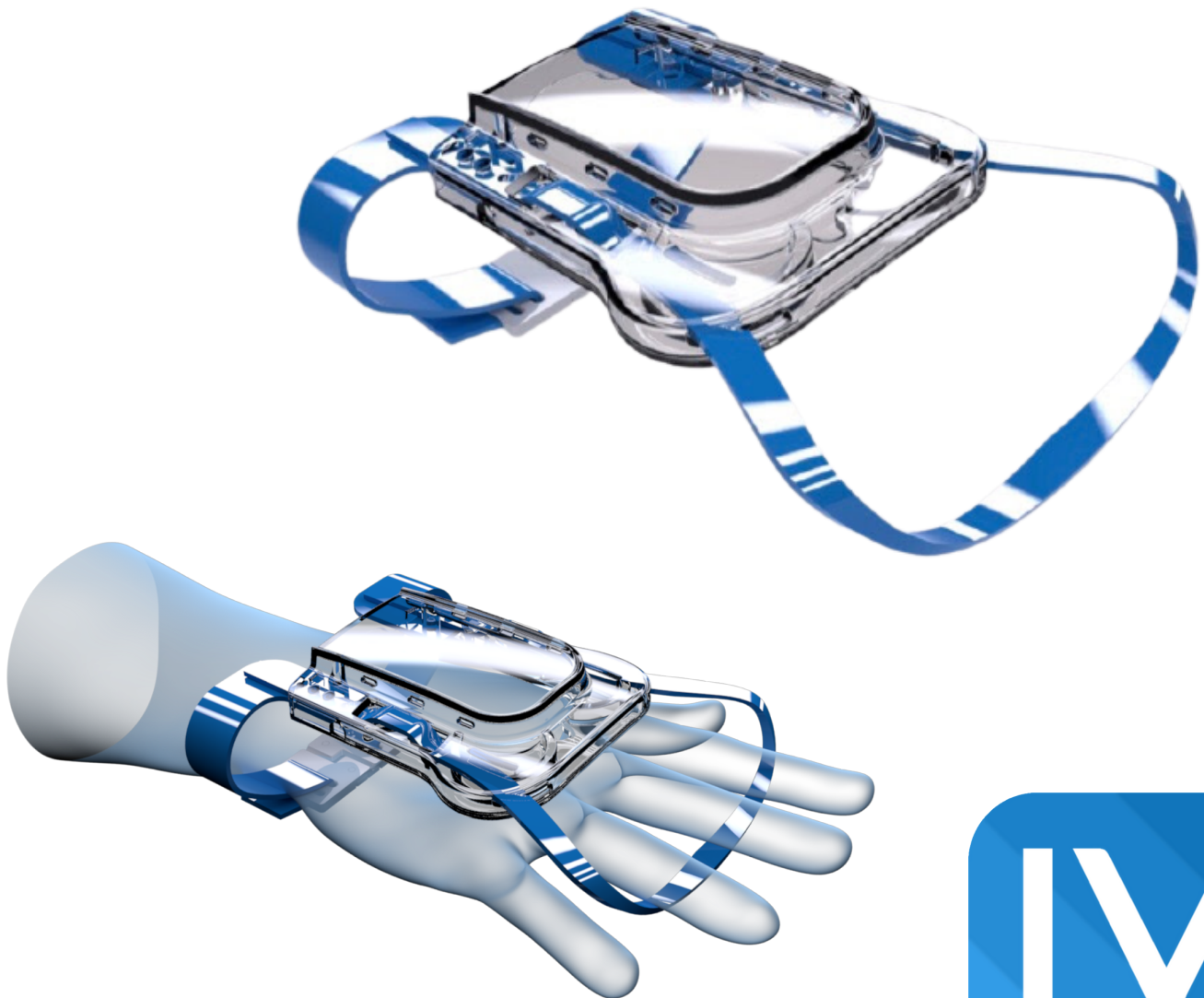


# FEATURES

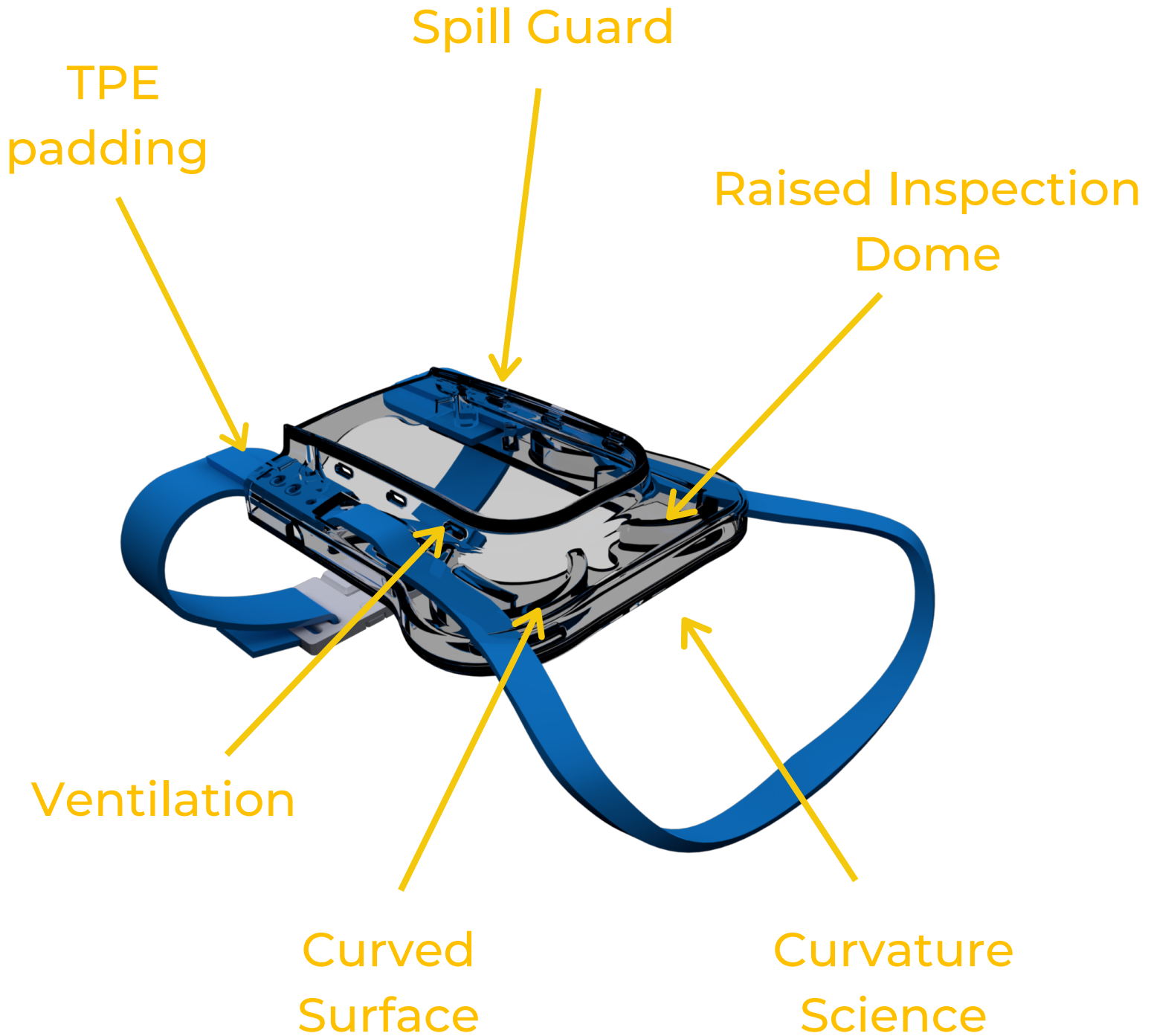



# SPECS

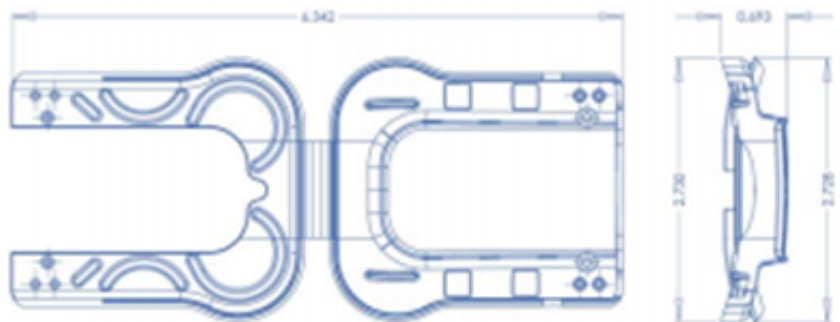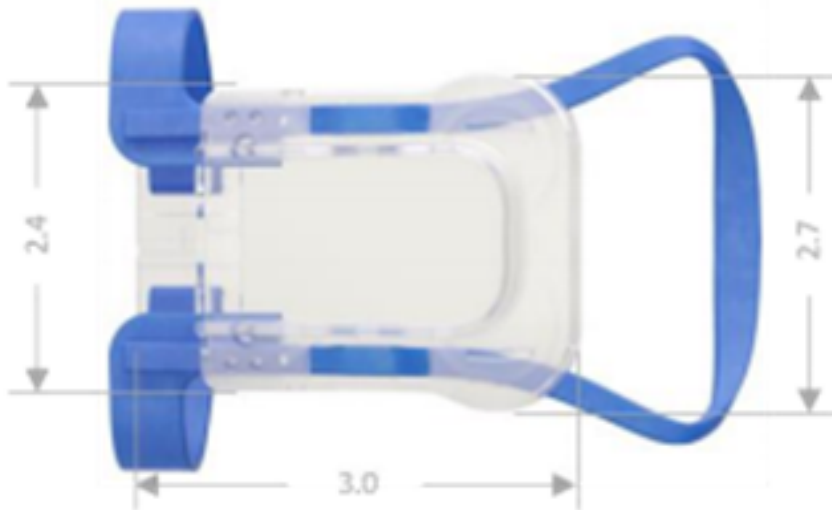

# INSTRUCTIONS

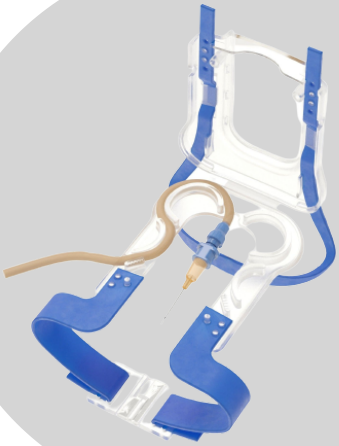

- Perform standard IV insertion as per hospital protocol
- Cover insertion site with medical tape
- Slide finger loop over patient's index to ring fingers
- Adjust finger loop for IV Glove stability and patient comfort
- Fasten wrist strap clasp, adjust to ensure comfort and securement
- Carefully run catheter line around patented S-Clip curves
- Gently snap the protective dome closed
- If necessary, trim excess strap length as needed

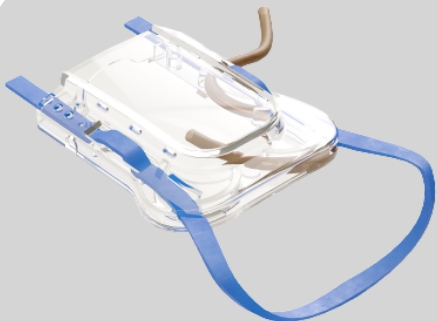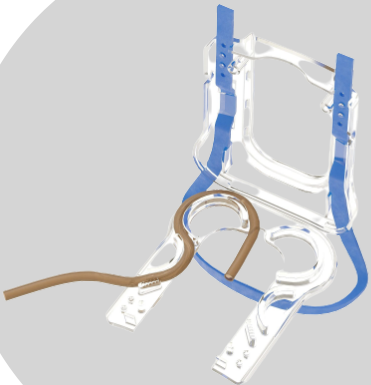

The IV glove can be used during the patient's stay based upon inspection and approval of the health care provider (ie. no damage).