

**[CLICK HERE FOR SHORT VIDEO SHOWING HOOD REMOVAL](#)**

1. With two people on opposite sides of the system, grasp the bottom edge of the hood to lift. To evenly distribute the weight of the hood, place hands approximately 15-20 inches from each end of the hood.
2. The hood should gently lift upward and forward until the hinge slide pins are free. *If the slide pins resist movement, gently rock the hood side to side while lifting upward.*
3. When slide pins are disengaged, carefully tilt hood forward and lift off the hinge. Move the hood to a safe area.



Installation of the hood in reverse order.